



HEAT SEALED POWDER COAT STEEL 10 BRONCHOSCOPE DRYING CABINET, KEY LOCK

SureDry
SCOPE CABINETS

Scopes not included

SC5430DRDP

Features:

- Meets or exceeds Joint commission, AORN, SGNA, and AAMI guidelines.
- Positive pressure fully redundant fan system
- Industries only unit with 2 intake and 2 exhaust fans
- Fan run time is fully adjustable and activated each time cabinet is closed
- HEPA filter removes 99.7% of particulates, .03 Microns
- Industry first replaceable HEPA Filter system without removal of scopes
- Holds 10 bronchoscopes with secure adjustable mounting system
- Medical grade electrical system
- Tempered glass double doors
- Easy to clean removable drip tray
- Multiple locking options available including badge entry
- Overall cabinet dimensions: 67.75" H x 30" W x 24" D
- Stainless steel or heat sealed powder coat low carbon steel, available in over twenty color options
- View colors online:
<https://www.harloff.com/finish-options/>

The Sure-Dry Scope Cabinet includes the industry best Positive Pressure HEPA Filtered Drying system. Inside air is replaced inside the cabinet 4 times per minute.

Optional dimmable LED lighting also available!

HARLOFF
INNOVATING STORAGE FOR CARE

Engineered & Manufactured in the U.S.



View similar products online at
<https://www.harloff.com/suredry-scope-cabinets/>

Toll Free: (800) 433-4064 • medicalsales@harloff.com
www.harloff.com • Ph: (719) 637-0300



Why is a positive pressure system important?

The unique design of the SureDry Scope Cabinet includes both top fans for intake and lower fans for exhaust. This push/pull design creates a vortex of air that flows through the cabinet ensuring maximum drying efficiency.

Intake:

Intake airflow comes through separate HEPA filters (which remove 99.7% of particulates, .03 Microns) driven by fully redundant fans producing a positive pressure vortex from the cabinet top. This creates an even circular motion of airflow over all scopes.

Exhaust:

The lower exhaust fans pull the circulated air out of the cabinet ensuring the flow of air is consistent from top to bottom, providing a complete drying system for the entire scope. This push/pull multiple fan system (an industry first) ensures a positive pressure system to meet all major guidelines and it ensures forced drying and equal drying levels across the full length of each and every scope.



SureDry Cabinets: A Smart Choice

to Meet and Exceed Current AORN, SGNA and AAMI Guidelines

Scope Storage Guidelines	Forced Air Drying	HEPA Filtered Air	Hang Scopes Vertically	Scopes Must Hang Freely, No Touching or Coiling	Easy to Clean and Disinfect	Closed Door Cabinet	Positive Air Pressure	Circulated Air Environment	Connectors — Use Disposables or Reusables Stored w/Scopes	Dry Scopes Completely	Uses Channel Drying (with Dri-Scope Aid®)
ANSI/AAMI ST91:2021	SureDry scope cabinets, with or without Dri-Scope Aid®, meet and exceed updated guidelines										
AORN	no data	✓	✓	✓	✓	✓	✓	✓	✓	no data	no data
SGNA	✓	✓	✓	✓	✓	no data	no data	no data	✓	✓	no data
AAMI	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓

10 Reasons to Choose **HARLOFF** for Your Scope Cabinets

A Smart Choice to Meet & Exceed Current SGNA, AORN and AAMI Guidelines



1 **Industry Best Positive Pressure HEPA Filtered Drying System.** Designed to add Dri-Scope Aid® now or later with easy “Plug & Play installation.

Proudly Made in the USA



6 Harloff products are proudly **Designed, Engineered and Manufactured in the United States** at our Colorado Springs, CO facility.


LED Lighting Option



2 **Brighten your cabinet** and more easily inspect scopes with fully dimmable optional overhead LED lighting which triggers on/off when you open and close the door.

Easy Access to HEPA Filters

Change HEPA filters **without opening the cabinet.** Save time and ensure no cycle interruptions.

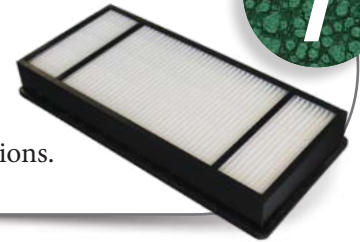

Keyed or Keyless Locks

Options include key locks, basic electronic push button locks, advanced keypads and proximity readers. View locking options online at www.harloff.com/locking-options/



Disposable Connectors

Save time and ensure compliance! Use single-use connectors for all internal channels of the endoscope. Part of the Dri-Scope Aid® package.



Hooks and Holders for Specialized Scopes

Major brands to include **Fuji, Olympus, Pentax, Karl Storz and others.**



Finish Options and Colors

Stainless Steel, Heat Sealed Powder Coat (High Gloss or Textured Low Gloss inside & out, view online at www.harloff.com/finish-options/) or upgrade to an **antimicrobial finish.**

12 Year Warranty

Our Manufacturer's Warranty is Unmatched in the industry. We build our cabinets to last and we stand behind them.

Mobile Scope Cabinet Kit

Convert any size of scope cabinet into a mobile version. Includes 4 3" casters and steel mounting plate.

