


# HEAT SEALED POWDER COATED JET CART SCOPE DRYING SYSTEM

STC-AL6-JET (6-scope capacity)  
STC-AL8-JET (8-scope capacity)

DATE ISSUED:  
April 2024

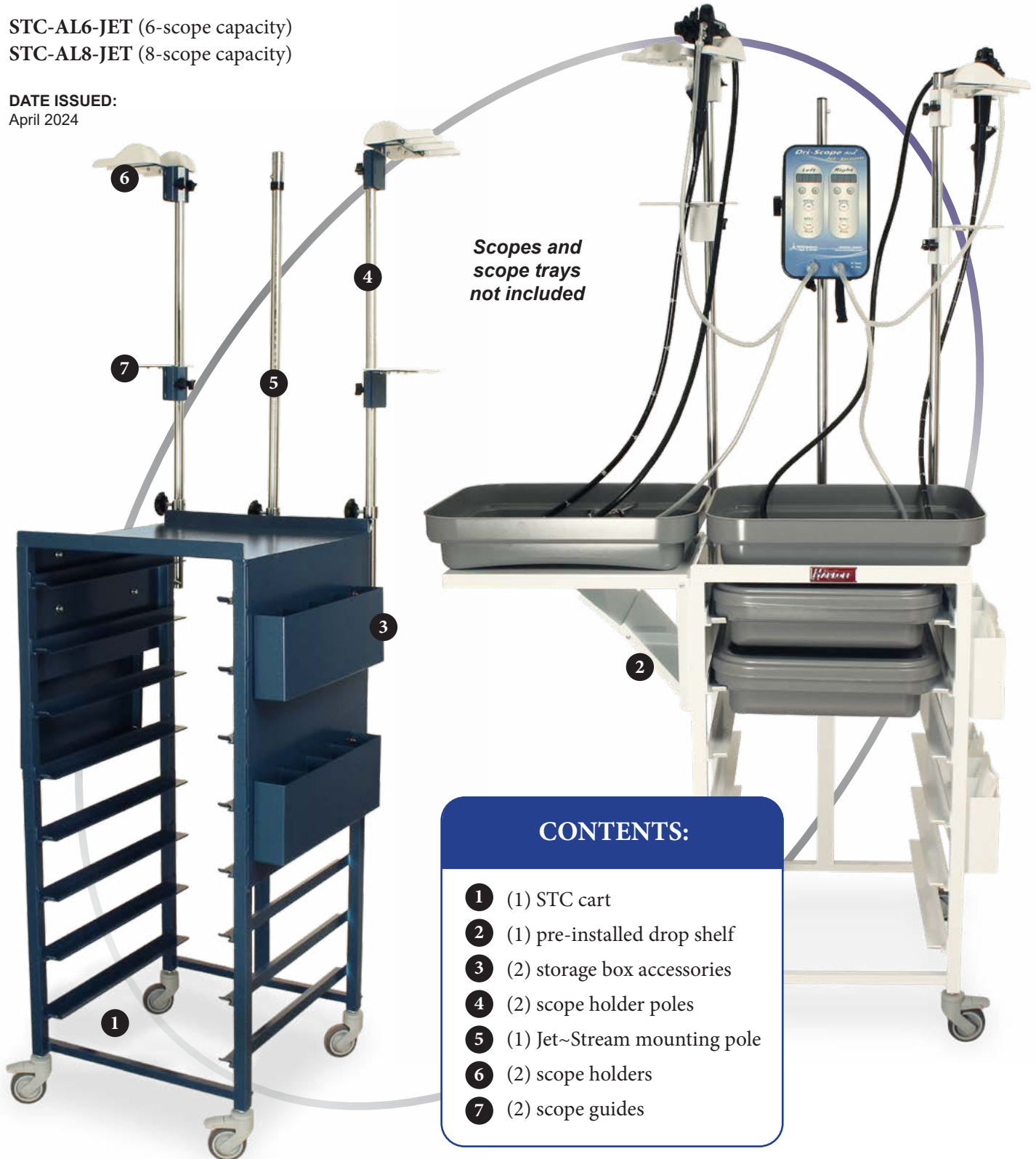

## CONTENTS:

- 1 (1) STC cart
- 2 (1) pre-installed drop shelf
- 3 (2) storage box accessories
- 4 (2) scope holder poles
- 5 (1) Jet~Stream mounting pole
- 6 (2) scope holders
- 7 (2) scope guides

**HARLOFF**  
INNOVATING STORAGE FOR CARE


**User Manual  
& Instruction Guide**

# Jet Cart Installation

## Installation:

Jet Carts come pre-installed with a drop shelf, accessory mounting screws, scope holders/guides and mounting poles set in the low position.

1. All mounting poles, scope holders and scope guides are adjustable and use compression screws to keep them in place. Unscrew to loosen or screw the handles in to lock in place. Poles extend or lower once these screws are loosened. The Jet~Stream unit also uses a compression screw with a handle for adjustment up or down.
2. Unscrew each adjustment handle and extend all three mounting poles to their top position to permit full operation.
3. Scope holders typically mount at the top of each mounting pole. Scope guides mount anywhere in the middle depending on your preference. Guides help separate scopes from the umbilical and other surfaces, helping to keep scopes sterile while drying.
4. Jet~Stream units mount on the middle pole at your desired height.
5. Endoscopes mount on scope holders with the valves at the front with the insertion tube hanging down through the “forks” of the holder. The umbilical cord/connector should hang down the back. Scope holders can be mounted on the poles facing any direction that works best for your setup.
6. The image at the right shows a typical drying setup. The inset images show alternative setups where scopes remain in the tray. Select a setup that meets your guidelines.
7. Please consult your Jet~Stream documentation for setup of tubing and for unit operation.

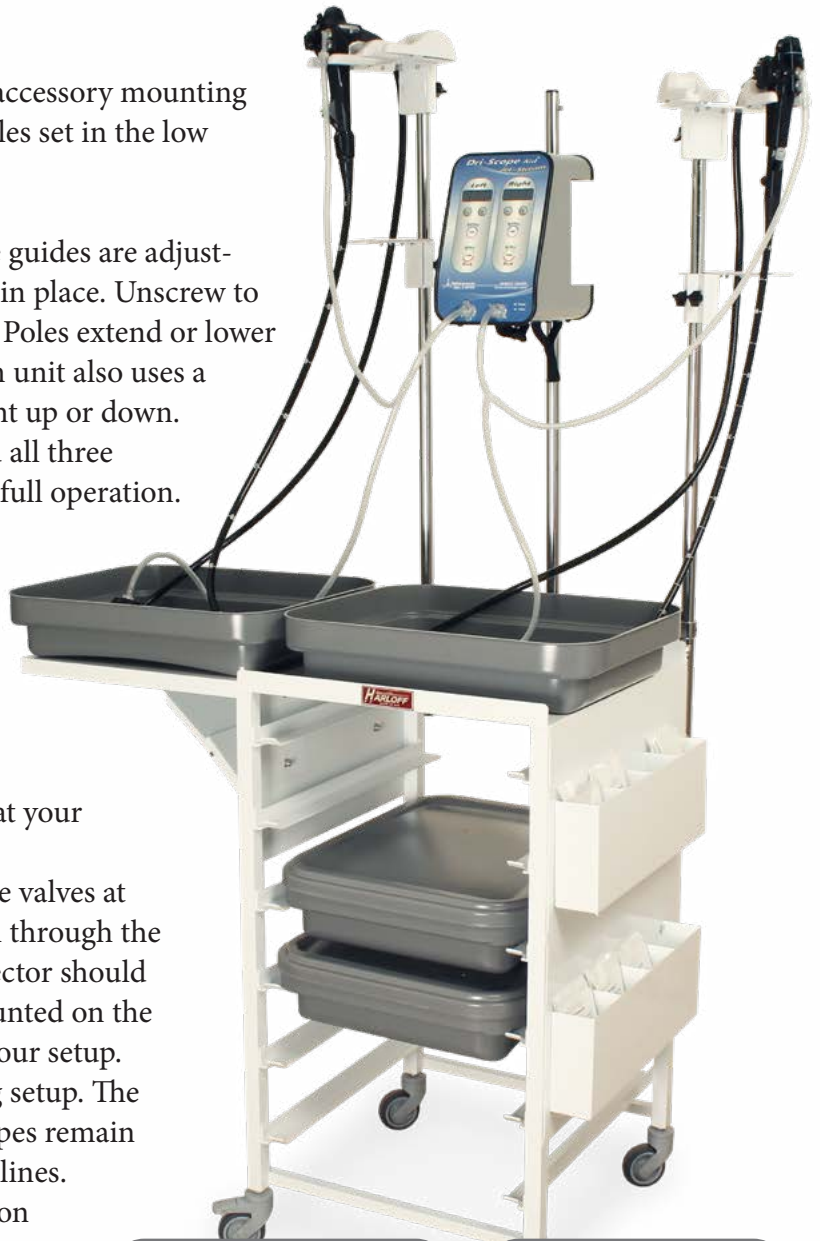

## Accessories:

1. To operate the drop shelf, pull open by lifting the shelf up. It will automatically lock into an extended position. To retract the shelf, slightly lift it with one hand and push the “elbow” area under the shelf with your other hand.
2. To mount your storage boxes, simply hang them on the mounting screws on the non-shelf side of your STC cart.

